USER MANUAL

BEREG-Horse-Kit Urine or Blood Sample Container

ENGLISH, PAGE	2
DEUTSCH, AB SEITE	7
FRANÇAIS, DE LA PAGE	12
ESPAÑOL, A PARTIR DE LA PÁGINA	17

Berlinger Special AG

Mitteldorfstrasse 2
9608 Ganterschwil
Switzerland

Tel. +41 71 982 88 11
info.special@berlinger.com
www.berlinger.com

TABLE OF CONTENTS

WHAT IS INCLUDED.....	2
INTENDED USE	2
HANDLING INSTRUCTION.....	3
WARNINGS AND PRECAUTIONS.....	5
ADDITIONAL HELPFUL MATERIALS	5

WHAT IS INCLUDED

In this **BEREG-Horse-Kit**, you will find:

- Either one (1), two (2) or three (3) orange and/or blue plastic sample containers with lasered number.

Each container with its own security cap and each sealed with a shrink sleeve wrap.

- Small labels with number and barcode.
- Watertight plastic bag/s, each with an absorbent pad, for the purpose of transportation to the laboratory (in compliance with the IATA Dangerous Goods Regulations).
- Additional possible accessories: blood collection accessories, gloves, urine collection bag/s, mini grip bag for documents, security bag/s.

Each BEREG-Horse-Kit is packed and sealed in:

- A cardboard box, which is sealed with a printed shrink-wrap.

Each BEREG-Horse-Kit is uniquely numbered, with the number shown on all its components: the plastic sample containers, the security caps, the cardboard box, and additional labels.

INTENDED USE

BEREG-Horse-Kit is intended to be used to collect, store, and transport animal urine or blood samples for the purpose of anti-doping and drugs of abuse testing outside of a medical setting and not on order of a health care professional.

This kit is not intended to be used in a medical setting nor on the order of a health care professional. This kit is also not indicated for or in assistance of disease screening or diagnosis.

Please note: The manufacturer does in particular not assume any liability for the application of incomplete or defective kits or any unlawful actions during the entire doping control procedure nor if the sample is placed in other containers or bags.

HANDLING INSTRUCTION

PREPARATION

Before opening the packaging, please check if the kit is properly sealed or shows any signs of manipulation:

- The shrink wrap of the cardboard box is intact.

To open the packaging, pull the wrap apart at the perforated line.

As soon as the kit is open, please check the following:

- The content of the BEREG-Horse-Kit is complete.
- The shrink sleeve wrap of each sample container is still intact before removing it.

All components of the BEREG-Horse-Kit have the same unique number on:

1. The sample container/s
2. The packaging box
3. The security cap/s
4. The additional labels

Images are indicative. Handling remains the same for orange or blue labeled container/s.

1

2

3

SECURITY CAP

Make sure that all elements of the cap are in place:

1. White plug
2. Spring clip
3. Black locking ring

1

2

3

RED RING

Please note: Remove the red ring before closure!

FILL LEVEL AND SCALE

A maximum of 4 blood tubes of 10 ml can be placed in each container.

The scale on the back of the plastic container indicates volumes of 25ml steps, from 25ml up to 190ml.

Attention: Do not fill the container above the horizontal ridge, which is indicated in the picture by a line marked "max.!"

HANDLING INSTRUCTION

HANDLING THE BEREG-HORSE-KIT

As soon as sufficient urine/blood tubes have been collected (in accordance with your regulations):

- Pour the urine into the container/s according to regulations or insert the blood tubes into the container/s according to regulations.
- Close each sample container by gently pressing its cap down on it while turning the cap at the same time in a clockwise direction. This turning should produce a few audible clicks and should be continued until the cap can be turned no further and is connected to the container without any gaps.

Attention: Please turn gently and do not use too much force!

INSPECT THE CLOSED CONTAINER

Please do the following:

1. Try to turn the cap counterclockwise, to ensure that the cap is securely closed.
2. Try to lift the cap off, to ensure that it cannot be removed.
3. Turn the sample container upside down, to ensure that there is no leakage.

PREPARATION FOR TRANSPORTATION TO THE LABORATORY

- To comply with the IATA DGR packaging regulations for the transportation of “exempt animal specimens”, the BEREG-Horse-Kit contains a watertight plastic bag with an absorbent pad for each sample container.
- Each sealed container must be put into a watertight plastic bag (leaving the absorbent pad inside the bag).
- Each bag must then be sealed by removing the backing paper from the adhesive tape and pressing it firmly together. Please also ensure that this action leaves as little air as possible in the bag, by stroking the bag upwards before closing it.
- After this procedure, please place the sealed container/s in the box and close it by putting the flaps back into the box.


WARNINGS AND PRECAUTIONS


- After sealing the samples, this product requires the handling of animal specimens. It is recommended that all animal-sourced materials be considered potentially infectious and handled with appropriate biosafety practices.
- Please ensure that all national and international regulations are being complied with when shipping the samples.

- Do not use the product if the package is damaged or the seal is broken.
- Discard unused, damaged, or leaking kits in accordance with local, state, and federal regulations.
- Do not use incomplete or defective kits or if package is damaged.
- Do not use the product beyond its expiration date (use-by).
- All items included in the BEREG-Horse-Kit are NOT provided sterile.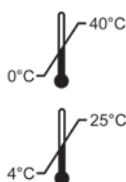

- Storage conditions of the kit before use: between 0°C and + 40°C.
- Storage conditions of the kit before use, when blood collection accessories are included: between +4°C and + 25°C.
- Maintain proper storage conditions during specimen shipping to ensure the integrity of the specimen.

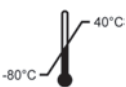- Long-term storage conditions of containers according to regulations: The equipment must be suitable for long-term storage in a frozen state, i.e. -80°C. Of the contents of an entire BEREG-Horse-Kit, only the collection containers that have been properly filled and sealed within the expiry date meet these requirements.

- All items included in the BEREG-Horse-Kit are single-use items.
- Do not re-use any item.

- Optimal testing performance requires adequate specimen collection and handling. Ensure the outside of the transport packaging and collection cups are not contaminated.
- Take care to avoid cross-contamination during the specimen handling steps. Specimens can contain pathogens. Ensure that containers do not contact one another, and discard used materials without passing over the containers. If gloves come into contact with specimen, change gloves to avoid cross-contamination.

ADDITIONAL HELPFUL MATERIALS

For more information on the product, consult the following resources:

The latest version of the user manual can be found on our website

www.berlinger.com/handling-instructions

For additional information, check out the product webpage

www.berlinger.com/berreg-horse-kit

For more information, including ways to contact Berlinger, check out our website

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