

Universal opener

to open BREG-KIT's
for urine and blood



Handling instruction assembling and use

Part 1: Assembling instruction

Supplied items

- black cup only for blood
- metal cap (urine/blood)
- 1 rubber pad
- 1 user manual

>> total weight approx. 12.0kg



1. Connect the machine to the electricity by plugging in the cable.

Information

- The standard power range is from 220 to 240 VAC. (optional 100 VAC)

Now the machine is ready to use.

Instruction of use

1st Step



Turn the power on by pushing the switch on the back side.



The green light on the front lights up.

2nd Step

Check first, if the bottle cap is fully and properly closed.

3rd Step



Place the BEREK-KIT small into the black cup
The BEREK-KIT urine is used without a cup

4th Step



5th Step

Put the metal cap on the top of the bottle.



Place the bottle and the blue metal cap onto the stand in the opening machine.

Please pay attention that the neck of the bottle is inserted exactly in the specified slot, until it stops.



Now press the green button, the opening process starts.

Important:

All is protected by the blue metal cap. You may take away the hands. It takes about 5 seconds

7th Step



Open the cap by turning it in an anti-clockwise direction.

Important:

Protect your hands by opening the cap with the rubber pad.

Please do not hesitate to contact us if you have any questions.

The manufacturer shall not held be liable if the BEREG-KIT for urine or blood and the universal opener are used beyond the manufacturers given limitations and handling instructions.

valid from March 2013 onwards