

Universal opener

for BREG-Horse-KIT




Handling instruction assembling and use

Assembling instruction

One compact opening machine is delivered together with cable and accessories as:

- black cup
 - one blue metal cap
 - 1 rubber pad
 - 1 user manual
- >> total weight approx. 12.0kg

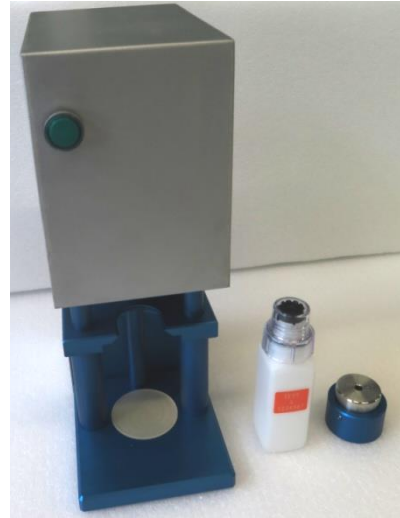

1. Connect the machine to the electricity by plugging in the cable.

Information

- The standard power range is from 220 to 240 VAC. (optional 100 VAC)

Now the machine is ready to use.

Instruction of use

1st Step



Turn the power on by pushing the switch on the back side.



The green light on the front lights up.

2nd Step

Check first, if the bottle cap is fully and properly closed.


3rd Step



Put the blue metal cap on the top of the bottle.



5th Step

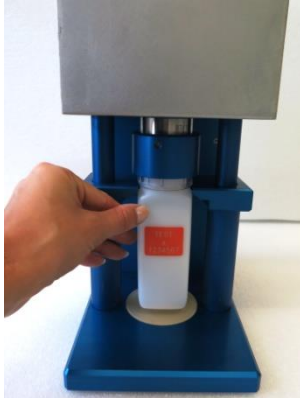

Place the black cup with the bottle and the blue metal cap into the universal opening machine.

Please pay attention that the neck of the bottle is inserted exactly in the specified slot, until it stops.



Now press the green button, the opening process starts.

Important:

All is protected by the blue metal cap. You may take away the hands. It takes about 5 seconds

7th Step



Open the cap by turning it in an anti-clockwise direction.

Important:

Protect your hands by opening the cap with the rubber pad.

The manufacturer shall not held be liable if the BEREG-Horse-KIT and the universal opener are used beyond the manufacturers given limitations and handling instructions.

valid from June 2016 onwards