



Handling instructions for the break cap (hand device) BEREG-KIT urine and BEREG-Horse-KIT

1. Check if the BEREG-KIT is properly closed by turning the cap in an anti-clockwise direction.



2. Place the metal break cap onto the security cap.



3. Place the glass bottle firmly into the bracket until it stops.

The rim of the security cap will be held by the ridge of the stand.



4. Use a hammer (not too small) and hit smoothly 1 time onto the metal cap.



5. Remove the metal break cap and open the separated security cap.

Note: To protect your hands from injuries, use the plastic pad to open the cap.

Now the sample is ready for analysis.

