

Manual compressor / handling instruction

- Put the metal break cap onto the kit.
 - >The blood kit additionally needs to stand in the black holder
 - > the urine and horse kit is placed directly into the stand in the opener. (no need of a cup for the urine/horse kit)
- Place the kit onto the stand (the rim of the security-cap will be held by the ridge of the opener, please see picture)

Metal cap for BEREK-KIT urine



Metal cap for BEREK-Horse-KIT



Metal cap and black holder for BEREK-KIT small



- Turn the wheel of the compressor in clockwise direction until you can hear a cracking noise and it stops. After that turn the wheel back again.
- Take out the kit, and lift the metal break cap. The separated security cap can now be opened. (to protect your hands from injuries, use the plastic pad to open the cap)



Important

Machine must be fixed with manual screw thread!

If you cannot fix the machine with screw thread please use the added "velcro" pads. (use the loop-part on the table and the hook-part on the machine)



Recommendation

Use once a month a drop of oil.

